




IDAL TIPS


BEFORE VACCINATION

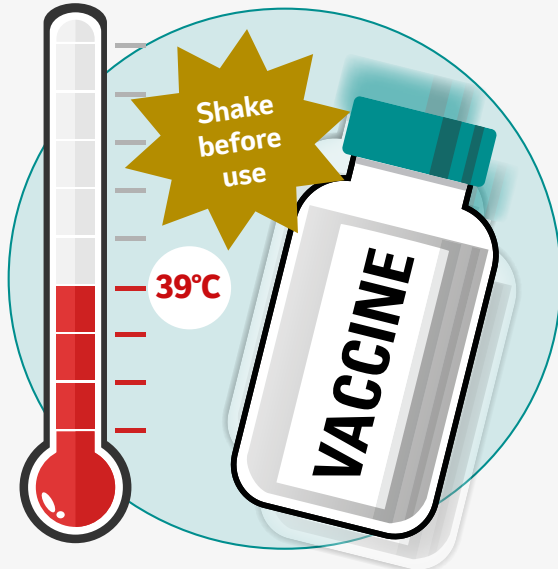

1 Warm the vaccine and possibly the solvent liquid (minimum 5 minutes) in lukewarm water.



2 Rinse the IDAL through at least 10 times. Change the bottles regularly. These can be ordered free of charge from your vet practice.

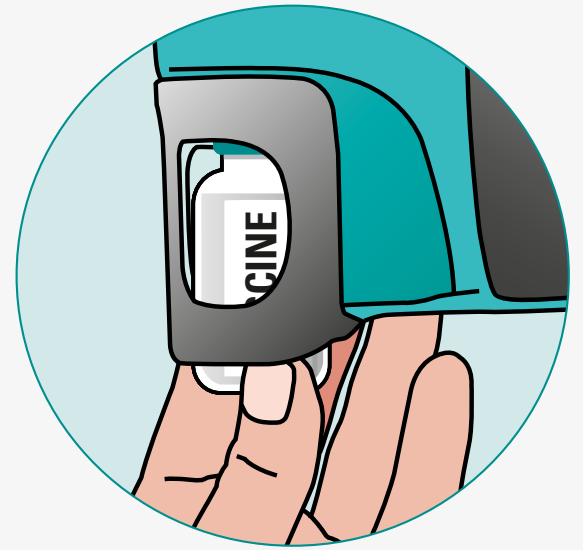


3 Place the vaccine in the IDAL. When doing so hold the IDAL upside down.

DURING VACCINATION



4 Reset the daily counter on the IDAL. The IDAL is now ready for use.

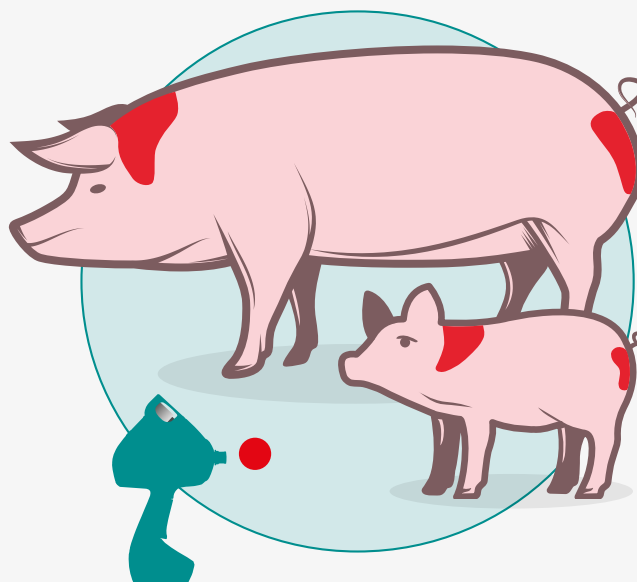


5 Give the vaccine at the correct location.




6 Administer the right amount of vaccine correctly into the skin. Ensure that there is sufficient contact time with the skin.

AFTER VACCINATION

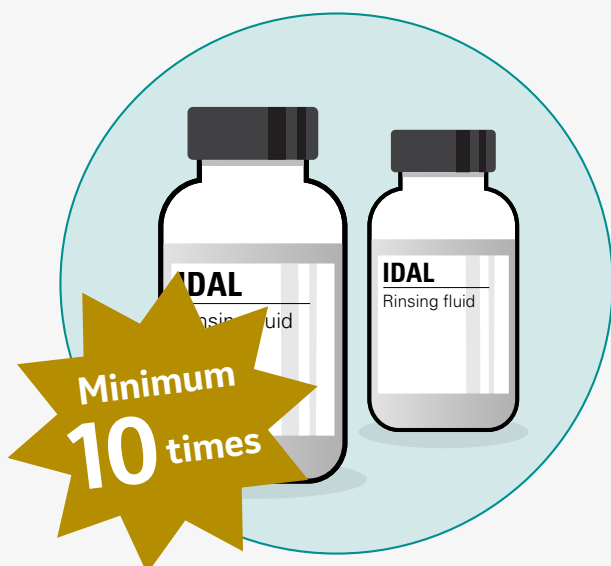

7 Rinse the IDAL again at least 10 times after use. Leave the bottle of rinsing fluid on the device

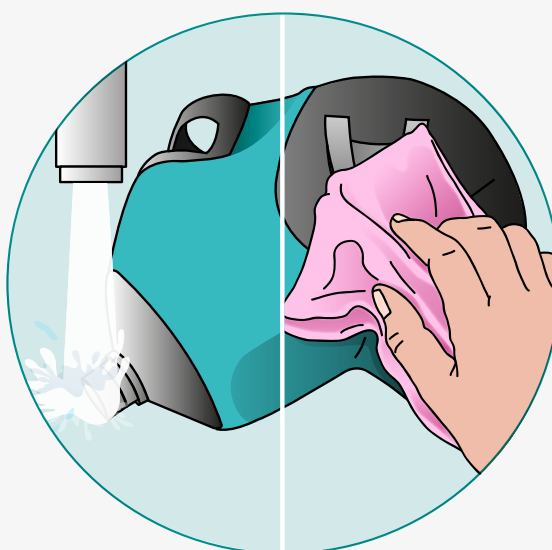

8 Clean the exterior of the IDAL. Only the head and the spike can go under the tap. Pay attention: The housing of the IDAL is only splash-proof.



9 Remove the battery from the IDAL and store the IDAL clean in the case. The IDAL is now ready again for the next use.

CHARGING THE BATTERY: 1.5 HOURS MAXIMUM



Scan the QR code for access to the IDAL users platform.

TECHNICAL QUESTIONS: +31 485 49 20 00